

## BRITEVENT®

---

### Product Data Sheet

---

**Product Description: Britevent Urine Temperature Indicators**

---

**Standard Range: 88 to 102°F or 32 to 39°C**

---

#### Construction:

Adhesive Tape	Double-coated Adhesive Tape
Covering Film	Polyester 3 mil stock
Color Change Material	Non toxic mixture of liquid crystal esters
Accuracy	±1°F / ± 0.6°C
Shelf Life	12 months from invoice date when stored at room temperature (i.e. 21°C (70°F) & 50% relative humidity).

---

**Application Data:** Apply temperature indicating label to outside of urine specimen cup. Read temperature after colors stop changing. Green is correct reading. If green is not visible, subtract 1°F from highest lit event.

**Function:** Display of temperature, as indicated by green event, of urine specimen cup. Label is available in direct read or 2°F offset forms.

**Use:** Ideal for use in clinics, hospitals, and doctors' offices.