

Thermostrip® WD

Washer Disinfection Temperature Indicator Label

Thermostrip® WD is a self-adhering, waterproof, color-changing label that measures the surface temperature of medical instruments in washer disinfectors, bedpan washers and any thermal disinfectors.



Using Thermostrip® WD on a medical utensil surface with each wash cycle will provide evidence that the proper disinfection temperature has been reached during washing.

Surface Temperature Disinfection Indicator



- **Dual Temperature Event Label 82 and 93°C**
- **Waterproof Irreversible Labels provide proof that disinfection temperature has been reached**
- **Labels change from white to black permanently when temperature has been attained**
- **Remove and retain as a permanent record of temperature attained**
- **Adhesive backed label 1-3/4" x 1/2"**
- **Accurate to ± 1°C of rated temperature**

Actual Size

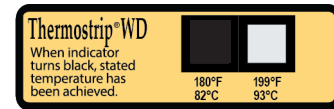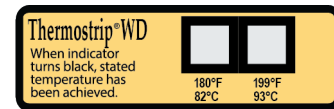

180°F Achieved

Specifications

P8001-50	Dual Temperature Event Label	180°F/82°C 199°F/93°C	1.47" × .51"
----------	------------------------------	--------------------------	--------------

How to Use

- Attach to the thickest clean dry medical utensil
- Place labeled utensil in washer disinfectant and run washer disinfectant
- Read label – Label will turn from white to black when surface temperature reaches rated temperature
- Remove label and attach to washer disinfectant documentation